
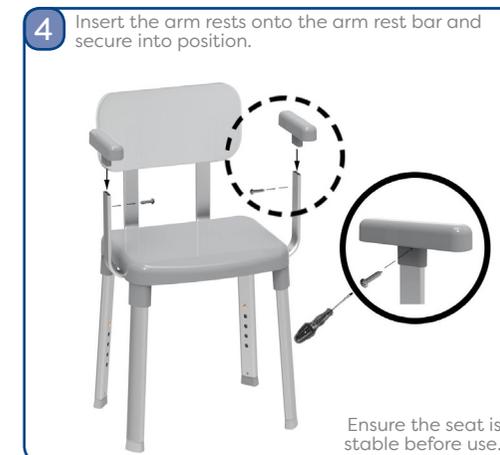
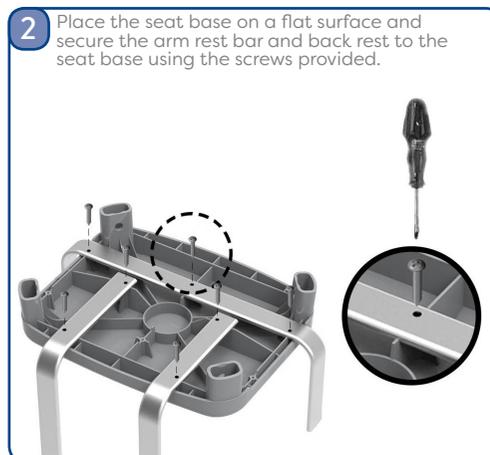
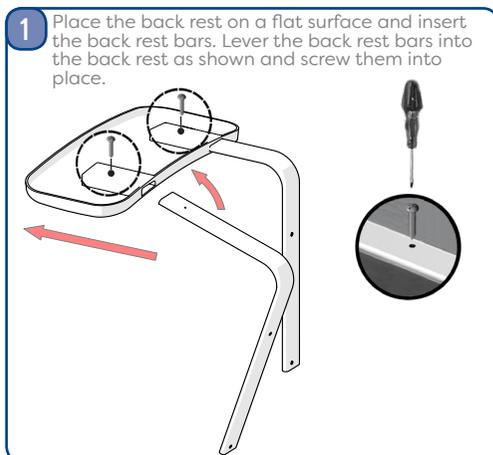


SAFETY & CARE

1. This product must only be used on a flat level surface.
2. **DO NOT STAND ON THIS PRODUCT.**
3. The maximum safe working load for the product is 100kg distributed evenly.
4. Always ensure the legs are correctly inserted and locked into the seat frame before use.
5. To retain the best quality finish, clean product regularly with a soft damp cloth.
6. Do not use abrasive or chemical cleaners, as these will damage the product.
7. Dry the shower seat after use.

AP130422

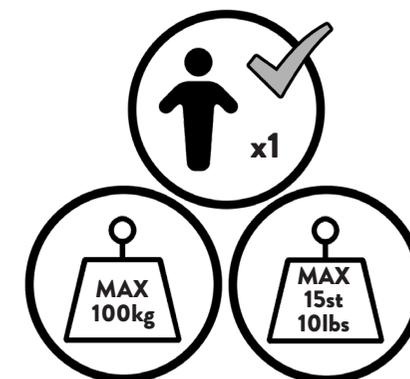

PRODUCT LOCATION

The adjustable seat must sit flat within the bathroom or within a shower tray. Curved surfaces of the bathtubs are not acceptable.

CAUTIONARY NOTE

Please ensure all legs are equal length, the seat is level and all pins are secured in position.

Take care not to extend leg beyond its furthest extension. If the leg is extended beyond this point please note that spring tension will propel the pin from the seat, so take care when adjusting the height.

If the pin and spring are removed from the leg, re-insert these components as shown in the picture to the right. Please note that the spring and pin need to be compressed in order to re-insert the leg into the seat.



Croydex
THINK BATHROOMS